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ANWENDUNG

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USER INSTRUCTIONS

WEARING METHOD

(the following methods shall be followed for each use)

- Hold the respirator in one hand with the nosepiece at the top. Press the respirator against your face with the nosepiece on the bridge of your nose.
- Place the bands behind your ears.
- Respect the laying direction (nosepiece at the top). The respirator should be worn covering the nose, chin and mouth. It should be applied tightly to the face.
- Using both hands, mould the nosepiece to the shape of your nose. Test the fit. Cup both hands over the respirator and exhale vigorously. If air flows around your nose, tighten the nosepiece. If air leaks around the edges, reposition the straps for a better fit. It is very important to press the nose clip firmly to make the protective mask close to the face. If not press firmly, will affect the performance.
- Once in place, do not handle the respirator as there is a risk of deterioration of it and contamination of the hands. If using the respirator against an infectious risk, practice simple washing or hydro alcoholic rubbing of the hands before and after each change of respirator.

USAGE LIMITATIONS AND WARNINGS

- When you get the respirator, you should first open it and hang it to a ventilated place for about 2-3 minutes.
- An increase in breathing resistance may indicate that the respirator are clogged with dust particles.
- The respirator is intended for use by one person only.
- Removeand/or replace the respirator immediately in case of difficulty in breathing, nausea, discomfort, Dizziness or other distress.
- Those diseases of respiratory system, such as asthma or emphysema, should consult with doctors and Receive physical examination before using protective respirator.
- Do not use when the respirator is damaged/deformed.
- Do not use if a correct fit of the face cannot be maintained. Please note this respirator it's for single Use respirator. Throw it awayafter one use. It is recommended to replace it after continuous use for 8 hours maximum.
- The lifespan of unused respirator is 24 months from the date of manufacture (if stored in its original box. In a clean, dry and uncontaminated place, between -2 and +50° C and with a humidity below 75 %).
- The respirator until use must be kept in the packaging to keep its properties.
- This product is not flame resistant and must not be use in areas with open flames and explosive Environment.
- This product is not suitable for harmful gases, steam, hypoxic environments and underwater operations.

IMPORTANT NOTICE

- This filtering half respirator is manufactured for COVID-19 protection only.
- This filtering half respirator is not a filtering half respirator for general use and shall not be used for purposes other than protection against COVID-19.
- This filtering respirator cover for the risk of COVID-19 contagion.**

Name: Filtering half mask to protect against covid-19
Model: NQW-01 (345105/5)
Standard: PPE-R/02.075 version 1
Manufacturing date: May 15th 2020
Shelf life: 2 years from manufacturing date
Manufactory: Guangzhou NaQing Environmental Protection & Technology Co. Ltd
Address: No 4, Second Industrial Street, Guannanyong, Shiji Town, Panyu District, Guangzhou, China
Importeur: Albert Kerbl GmbH
Adresse: Felizenzell 9, 84428 Buchbach, Germany www.kerbl.com

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